## LIMITED WARRANTY

### What does the Warranty Cover?

Pelstar LLC scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, prepaid and properly packed to Pelstar. If manufacturer determines that a defect of material or in workmanship exists, customers' sole remedy will be repair or replacement of scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

### Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

### What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instruction, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

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Contactez le Service Garanties au 1-800-815-6615.

Another innovation from the people at

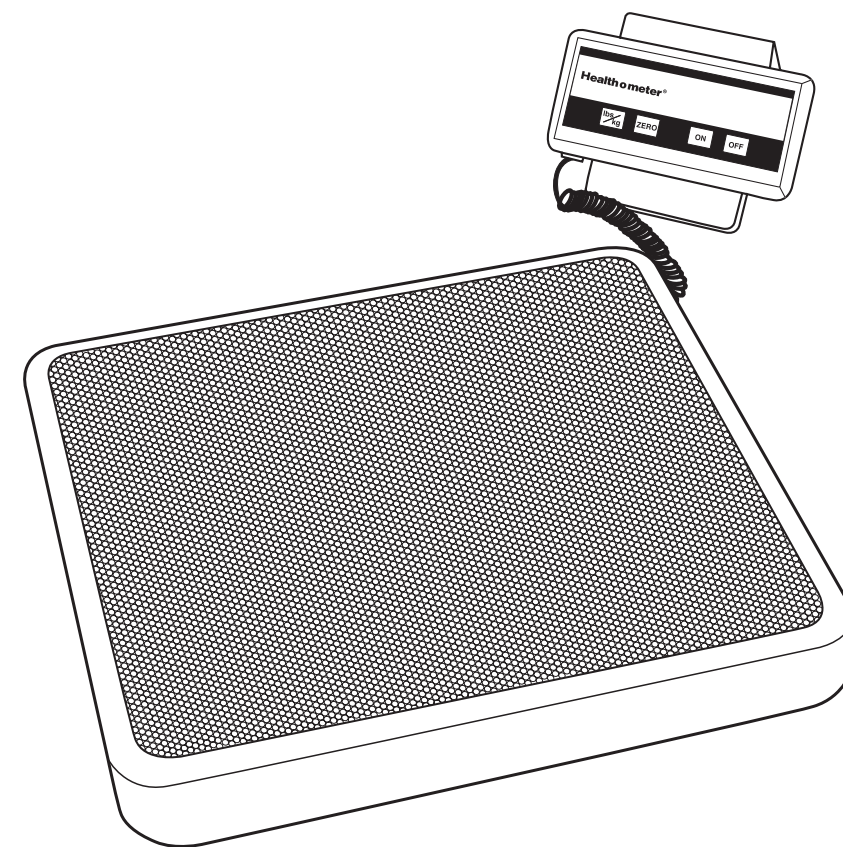
Health o meter®  
Professional

7400 West 100th Place, Bridgeview IL 60455 • 1-800-815-6615  
www.healthometermedical.com

Health o meter® Professional products are manufactured, designed and owned by Pelstar LLC.  
Health o meter® is a registered trademark of Sunbeam Products Inc., Boca Raton, FL 33431

Health o meter®  
Professional

## OPERATING INSTRUCTIONS INSTRUCCIONES DE OPERACIÓN MODE D'EMPLOI

## PHYSICIANS REMOTE READ DIGITAL SCALE

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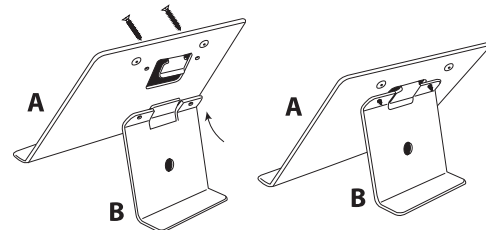
MODÈLE: 320KL / 349KLX

# PHYSICIANS REMOTE READ DIGITAL SCALE

MODELS: 320KL / 349KLX

old Part A (Stand) with the knobs facing away from you. Slide the center tab of Part B (Support) up and under the tab on the back of the Stand. Be sure to align the two parts, so that the tab on the Support can slide beneath the tab on the rear of the Stand. The holes on both parts of the bracket should now line up.

- Turn the bracket over so Part A is facing you and insert the two screws (included) into the holes and through the openings in Part B.
- Using a Phillips head screwdriver, tighten the screws to secure Part B to Part A.
- Align the holes in the back of the remote display head with the protruding knobs on the bracket. The remote head should slide down onto the knobs, firmly and securely. The display bracket is now ready for use.

## MAINTENANCE

**PROTECT YOUR SCALE FROM SHOCK, OVERLOAD, LIQUID SPILLS, EXTREME TEMPERATURES, OTHER ABUSES.** Preserve the finish of the scale by wiping only with damp, soft cloth and mild, nonabrasive detergent.

## SPECIFICATIONS

**Dimensions:**  
Remote Head: 5" x 3" x 1 1/4"  
Platform Size: 12" x 12 1/4" x 2 1/4"  
Power Source: 9 Volt battery or 110V AC adapter output: 9V DC 200 mA (included).

Model	Capacity	Resolution
320	400 lb / 180kg	0.5 lb / 0.2kg
349	400 lb / 180kg	0.2 lb / 0.1kg

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